

SWAB

reference guide

Bacterial Cultures



eSwab

eSwab Collection Device

To collect:

Remove cap from tube. Set tube upright on a firm surface. Collect specimen with the enclosed swab. Place swab in tube. Break swab shaft at red score line. Discard top portion of shaft. Screw cap tightly onto tube. Transport refrigerated.

FREQUENTLY ORDERED TESTS	TEST CODES
Aerobic/Anaerobic Culture, Swab	CWDANA
Aerobic Culture, Swab	CWND
Ear Culture	CEAR
Eye Culture	CEYE
Gram Stain for Bacterial Vaginosis only	GRAM
MRSA Surveillance Screen	CMRSA
Yeast Culture	CYEAS